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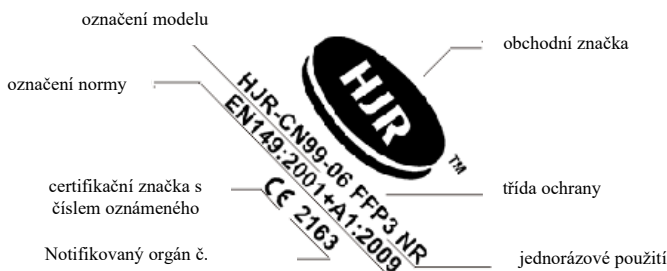
Popis výrobku:

Ochranná obličejová maska FFP3 NR

Model:

HJR-CN99-06

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Vlastnosti:

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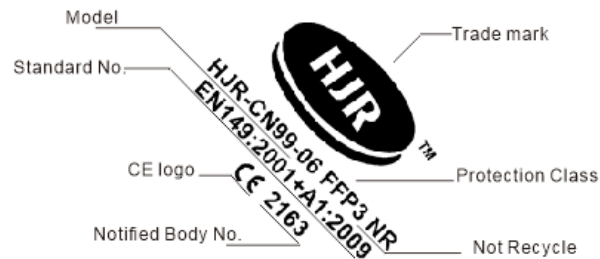
Meaning of the Product Marking

Features:

- Five-layer Filtering Structure
- High-quality and Skin-friendly Non-woven Materials
- Comfortable Nose Bridge and headband Design
- Without Breathing Valve
- Elastic and Soft head-band
- Protections against Air-born Particles, Pollutants, Dusts, etc.
- Light-weight, Breathable, Hypoallergenic, High-filtration-efficiency

Specifications:

- Color: White
- Dimension: 159.99mm*104.44 One Size
- Materials: Polypropylene Non-woven Fabrics
- Standard: EN149:2001+A1:2009
- Protection Class: FFP3
- Filtration Efficiency: $\geq 99\%$

This product is designed to provide adequate sealing on the face of the wearer against the ambient atmosphere, and protect respiratory system, it has passed EN149:2001+A1:2009 to show conformity to the EHSR of Regulation (EU) 2016/425.

Shenzhen HJR Electronics Technology Co., Ltd.

Fitting Instructions:

1. Put Bridge of nose up, use mask to hinder the chin.
2. Pull the top band over the top of your head and place it at a higher position behind your head.
3. Pull the headband over your head. Put it in below the back of the neck and ears.
4. Adjust the mask and cover it completely to close mouth and nose to fit face.

Packaging:

Packaging Material: Paper Box

Qty:50pcs/Box,400pcs/carton

Storage Condition:

The mask with sealed packaging should be stored in a cool & dry place, and away from direct sunshine.

Please seal the mouth of bag timely after opening.

Warning:

This product is non-medical and disposable.

Do NOT use the product exceeding the appropriate scope of use.

This product is not intended for long term use. A period of continuous wearing of no more than 4 hours is recommended.

Materials which may come into contact with the wearer's skin could cause allergic reactions to susceptible individuals. In case of any discomfort, please remove the mask from your face immediately and seek medical attention.

Spotted, scratched or damaged mask should be replaced immediately.

This product can filter certain pollutants, but cannot eliminate the risk of illness or infection. Incorrect use of this product may result in illness or even death.

It is unlikely that the requirements for leakage will be achieved if facial hair passes under the face seal

Shelf Life: 3 Years

Notified Body:

Name: Universal Certification and Surveillance Services Trade Co.

Address: Necip Fazil Bulvari Keyap Sitesi E2 Blok No.44/84 Yukari Dudullu Umraniye, Istanbul/Turkey

Identification Number:2163-PPE-772

Manufacturer:

Shenzhen HJR Electronics Technology Co., Ltd.

Address:

5/F, Building A3, Xinjianxing Science and Technology Industrial Park, No. 3333, Guangqiao Avenue, Gongming Street, Guangming New District, Shenzhen City, Guangdong Province, China.

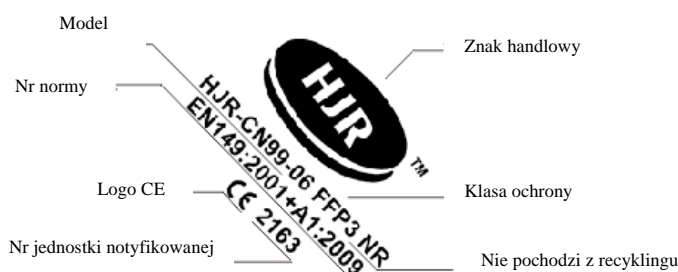
Declaration of Conformity on Website: www.batist.com

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